

# SUPERTOUC<sup>®</sup>



## PRODUCT INFORMATION

### FFP3 Folding Respirator

#### Description:

With a 3-panel fold flat design engineered to fit a wide range of face shapes and sizes, this FFP3 respirator mask offers ultimate comfort. A highly efficient valve allows warm air to escape more easily while providing all the protection required of a P3 respirator.

- ✓ CE Certified to EN 149:2001+A1:2009
- ✓ Filter efficiency level greater than 99% (FFP3)
- ✓ Valved to avoid heat build up for greater comfort
- ✓ Category III
- ✓ Latex free elasticated head strap
- ✓ Adjustable nose bridge for comfortable fit

#### Packaging:

10 Pieces per inner - 20 boxes per case

#### Product Codes:

SPE-FP3V Standard FFP2 Folding Respirator

#### Standards:



0086 CAT III



Single use only  
Do not reuse


PPE Regulation (EU) 2016/425 and personal protective equipment by the PPE Regulation 2016/425 as amend to apply in GB.

EN 149:2001+A1:2009 - Respiratory protective devices:

## SUPERTOUC<sup>®</sup> How To: Don & Doff Respiratory Protective Equipment

### Part 1 Correctly fitting (donning) Respiratory Protective Equipment

1.

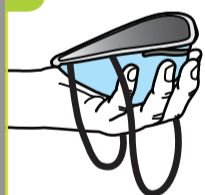

Cup the respirator in one hand with the straps hanging downwards. Check the respirator for any damage. If it is defective, **do not use it** and dispose of it immediately.

2.



Hold the respirator in place on your face. Make sure the nose piece is facing up.

3.



Pull the bottom strap over your head and position it at the bottom of your neck, below the ears. Do the same with the top strap, positioning it at the crown of the head above the ears.

4.



Tighten the straps as desired, and mould the metal nose strip with your fingertips to ensure a secure seal around your face.

5.



To ensure that the mask has no leakage, cup your hands over the filter material and breathe in. If the respirator sucks down hard onto the face, the seal is secure.

6.



Continue to breathe in and out sharply. If the respirator is fitted correctly, you should not feel any leaks around its edges.

### Part 2 Correctly removing (doffing) Respiratory Protective Equipment

1.



**Do not** under any circumstances touch the front of the respirator as it may be contaminated.

2.



To remove, pull the bottom strap over your head, followed by the top strap. Hold onto the top strap, using this to pull the respirator away from your face.

3.



Carrying the respirator by the top strap, discard it into a waste container as soon as possible.

4.



Make sure to wash your hands thoroughly before touching any other surfaces or objects.

www.supertouch.com  
sales@supertouch.com  
0345 130 9922

Stay Clean. Stay Safe. Stay Alert.

The information shown on this specification sheet is intended as a guide and is advisory only. All details were correct at the time of issue. As conditions of use are ultimately beyond our control, we advise that the product is tested for the particular application before use.

Issue No.: 1

Issue Date: 30/01/2023